Medtrum EasyView Website

Korisničko uputstvo

Humalog® je trgovačka marka Eli Lilly i Kompanije.

NovoRapid® je trgovačka marka Novo Nordisk A/S.

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
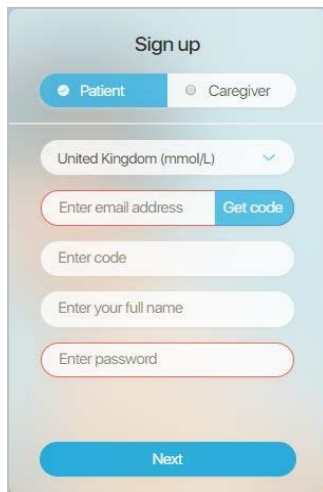
The image shows a mobile app sign-up screen. At the top, it says 'Sign up'. Below that are two radio buttons: 'Patient' (selected) and 'Caregiver'. Then there is a dropdown menu showing 'United Kingdom (mmol/L)'. Below the dropdown are four input fields: 'Enter email address' (with a 'Get code' button next to it), 'Enter code', 'Enter your full name', and 'Enter password'. At the bottom is a large blue 'Next' button.

1. Odaberite **Patient** (Pacijent).
 2. Odaberite Vašu zemlju/regiju sa liste.
 3. Unesite validnu email adresu kao naziv Vašeg naloga.
 4. Kliknite na **Get code** (Traži kod) i dobićete šestocifreni verifikacioni kod na email od Medtrum tima. Molim Vas da provjerite neželjenu poštu u slučaju da je email otišao tamo.
 5. Unesite šestocifreni verifikacioni kod. Vodite računa da imate 10 minuta da unesete kod prije njegovog isteka, i ukoliko kod istekne moraćete da tražite novi verifikacioni kod i da unesete poslednji kod koji Vam je poslat.
 6. Unesite Vaše puno ime kako bi Vas vaš lijeekar lako identifikovao.
 7. Kreirajte šifru i zapamtite je.
- ✓ Šifra mora da sadrži karaktere iz tri od sledeće četiri kategorije:
- Velika slova abecede (A do Z)

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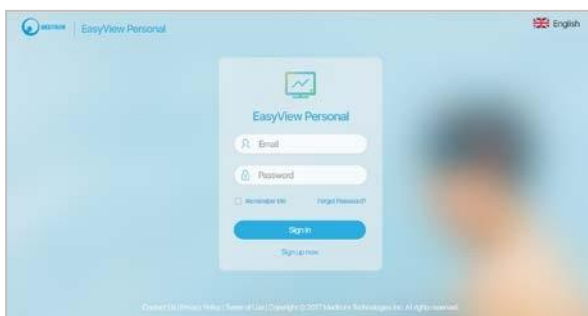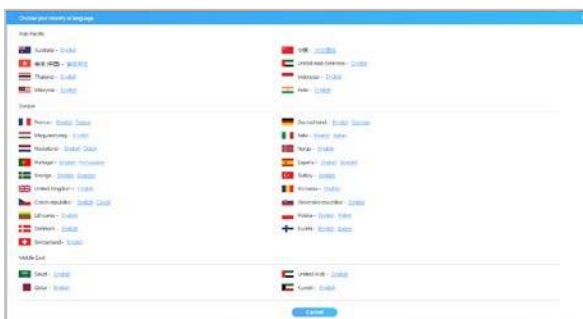
Report Settings | Data to view | Save | Cancel

Report Settings (mmol/L)

Show alerts: ☒ On

Glucose Target Range

Overall: 5.0 - 7.8

Hypoglycemia: 2.1

		Glucose Target Range		Time Period		Post Meal Analysis	
		Low	High	From	To	From	To
Breakfast	Before	3.7	7.2	07:00	08:15	1.00	3.00
	After	5.6	8.9				
Lunch	Before	5.1	7.2	12:00	13:00	1.00	3.00
	After	5.6	8.9				
Dinner	Before	3.9	7.2	18:00	19:00	1.00	3.00
	After	5.6	8.9				
Night-time		5.6	8.9	23:00	03:00		

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
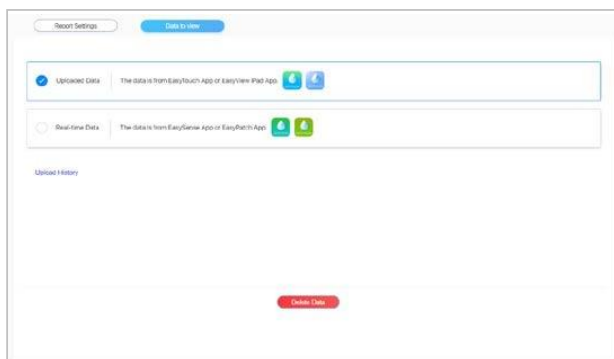
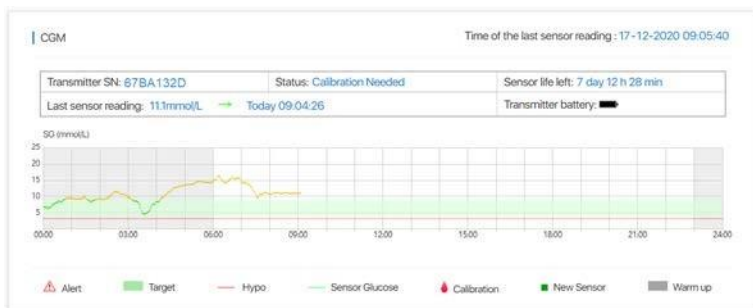
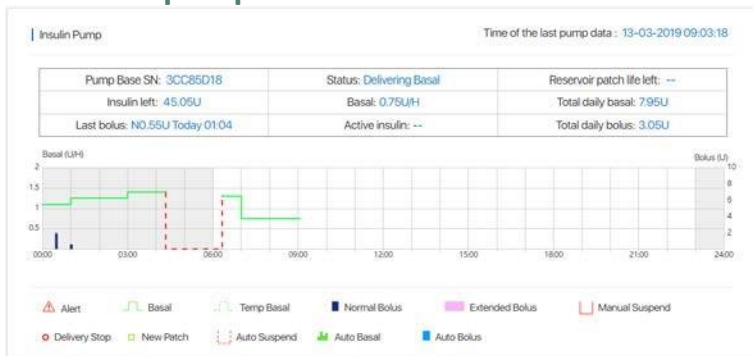
Izveštaji

Glucose Statistics									
BG		CGM		Insulin		Carbs		Exercise	
Average		Average		Average Daily Dose		Average Carbs/Day		Average time/Day	
9.4 mmol/L		9.1 mmol/L		37.33 U		44 g		--	
SD	#	SD	#	SD	# days	SD	#	SD	#
3.17	30	3.46	4901	2.57	4	18.6	6	--	--

Estimated A1C: 7.3% calculated from SG values

Note: Estimated A1C does not replace lab measurement

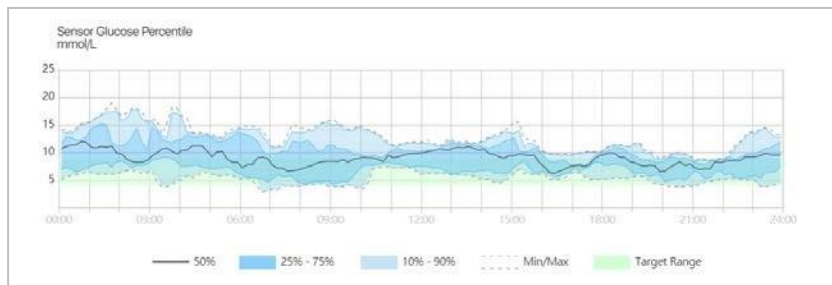Drugi dio prikazuje izvještaj u procentima, u kom možete da pregledate distribuciju SG u određenim trenucima u danu, tokom odabranih dana.

Treći dio izvještaja uključuje detaljne statistike o glukozu i isporučivanju insulina. Ovdje takođe možete da pregledate distribuciju glukoze i insulina u odabranim danima u obliku pita grafikona.

Možete da podesite period klikom na **Set** (Podesi), i potom odabirom trajanja i krajnjeg datuma.

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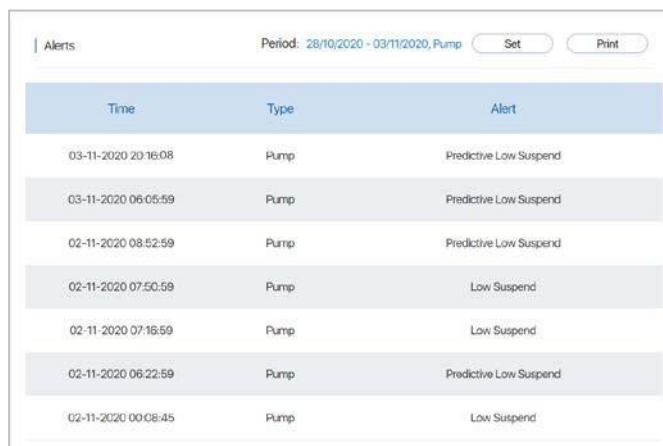
Apply Cancel

5.9 Pregled podešavanja uređaja

Dashboard Daily Summary Day by Day Overview Sensor Overlay Trend Analysis Events

Logbook Alert Device Settings Review

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Bolus

Maximum Bolus: 25U		Bolus Increment: 0.05U		Bolus Calculator: On	
Combo/Ext Bolus: On		BG Reminder: Off		Active Insulin Time: 3H	
IC Ratios(g/U)					
Start	48	--	--	--	
Carbs	15	--	--	--	
Insulin Sensitivity mmol/L					
Start	48	--	--	--	
BG	2.8	--	--	--	
BG Target					
From	48	--	--	--	
Range	5.6-5.6	--	--	--	
Bolus Reminder (Off)					
From	--	--	--	--	
To	--	--	--	--	

Izvjestaji

CGM

Glucose Alerts: On
Alert Silence: Off

Glucose Limits

From	48	--	--	--
Low	4.4	--	--	--
High	13.3	--	--	--

Predictive Alerts (h:mm)

High	Off	--
Low	Off	--

Rate Alerts (mmol/L/min)

Rise	Off	--
Fall	Off	--

Snooze (h:mm)

High	Off	--
Low	Off	--

Calibration

Calibration Reminder	Off	--
Calibration Repeat	01:00	
Graph Range	24H	

PCM

Audio Options: Audio+Vibrate	Patch Expiration: On 2H	Sensor Expiration: Off
Low Reserve: Insulin 10U	Auto Off: Off	Confidence Reminder: On
Hourly Max: 25U	Daily Max: 80U	Low Suspend: On 3.5mmol/L
Predictive Low Suspend: On 00:35 min	Out Of Range: --	

Time Format: 24H	Username: 0703054461	Display Settings (Screen): 60sec
Display Settings (Backlight): 60sec	Device Options (CGM System): On	Device Options (Insulin Pump): On
Glucose Unit: mmol/L	Alarm Clock: Off	Telephone: --

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Complications:
Create

Concomitant diseases	Date of diagnosis	Suggestions	Action

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Select reports to print:

☒ All


☒ Dashboard	From: 29-10-2020 To: 04-11-2020
☒ Daily Summary	From: 29-10-2020 To: 04-11-2020
☒ Day by Day Overview	From: 29-10-2020 To: 04-11-2020
☒ Sensor Overlay	From: 29-10-2020 To: 04-11-2020
☒ Trend Analysis	From: 29-10-2020 To: 04-11-2020
☒ Logbook	From: 29-10-2020 To: 04-11-2020
☒ Event Summary	From: 29-10-2020 To: 04-11-2020
☒ Alert	From: 29-10-2020 To: 04-11-2020
☒ Device Settings Review	From: 29-10-2020 To: 04-11-2020

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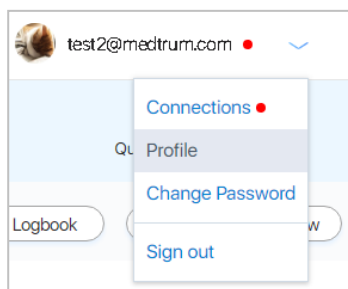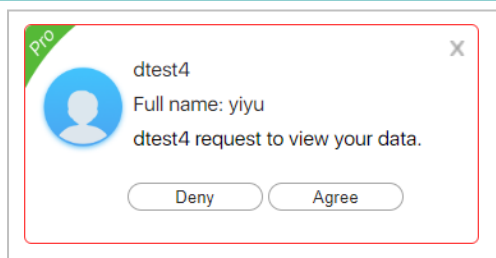
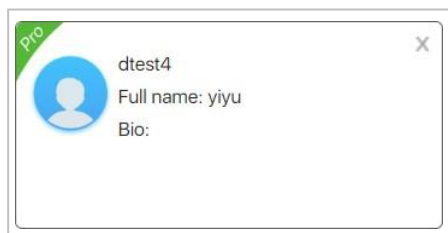
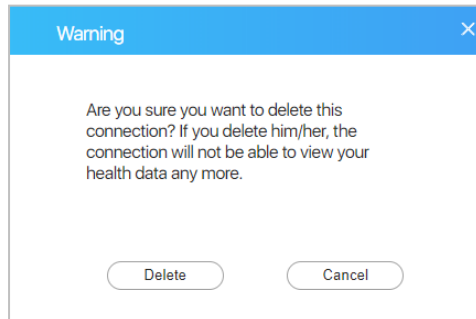
A form titled "My profile" with a "Logout" link in the top right. The form contains the following fields: Username (text, value: test10@medtrum.com), Profile photo (icon), Full Name (text, value: Clair Simpson), Gender (dropdown, value: female), Height (text, value: 167, unit: cm), Weight (text, value: 65, unit: kg), Date of birth (calendar icon, value: 1970-01-01), Type of diabetes (dropdown, value: Type 1), Date of diagnosis (calendar icon, value: 1970-01-01), Address (three stacked text fields), Postcode (text field), Mobile (text field), Telephone (text field), Email (text field), and Bio (text field). At the bottom right are "Save" and "Cancel" buttons.

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